

User's Manual


Bovine Leukaemia

Virus Antibody

ELISA

A monoclonal antibody-mediated complex-trapping-blocking ELISA to detect Bovine Leukaemia Virus p24 antibodies in serum samples of bovines and ruminants

REF B1111-AB01

 96

December 2020

Gebruik alleen de juiste versie van het protocol dat meegestuurd wordt met de kit.

Please use only the valid version of the package insert provided with the kit.

Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Arbeitsanleitung.

Si prega di usare la versione valida dell'inserto del pacco a disposizione con il kit.

Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.

Inhoud / Table of Contents / Inhaltsverzeichnis / Tabella die Contenuti / Tabla de Contenidos

1.	Introduction.....	3
2.	Intended use of the test kit	3
3.	Principle of the test kit	3
4.	Contents	4
5.	Handling and storage of specimens	4
6.	Wash protocol	5
7.	Preparations	5
8.	Test protocol Qualitative.....	6
	Before starting this test read “ preparations ”	6
9.	Test protocol Quantitative	8
	Before starting this test read “ preparations ”	8
10.	Precautions.....	9
11.	Validation of the test.....	10
12.	Interpretation of the test results.....	10
13.	Symbols used with EVL ASSAYS	11

1. Introduction

Serological identification of Bovine Leukaemia Virus (BLV)-infected cattle can be performed by screening for antibodies against a 24,000 Dalton polypeptide (p24) of the viral core. The agar gel precipitation test (AGPT), and conventional enzyme-linked immune sorbent assays (ELISA) have proven adequately sensitive and specific for antibody detection in serum and milk. However, more specific ELISA's are required to confirm results obtained by AGPT and conventional ELISA's.

2. Intended use of the test kit

This diagnostic test is intended to identify BLV-p24 antibodies in individual serum samples. This test which make use of monoclonal antibodies against P24 BLV antigen, this monoclonal antibody-mediated ELISA gives minimum non-specific reactions. This ELISA has a similar specificity to the agar gel precipitation test (AGPT), but is more sensitive.

3. Principle of the test kit

The test is based on the reaction of monoclonal antibodies to this end BLV P24 monoclonal antibodies have been coated to a 96-well microtiter strip plate. The serum is added to the wells together with BLV antigen to the monoclonal antibody coated plate.

- **Qualitative**
The serum sample is added (diluted 1:1) to the wells of the coated plate.
- **Quantitative**
The serum sample also can be titrated using a 3-step dilution, starting with undiluted (→ 1:3 → 1:9 → 1:27).

After incubation and washing, the bound monoclonal antibodies are detected by streptavidin HRPO conjugate.

Blocking of the color reaction in the wells is directly related to the presence of BLV p24 antibodies in the samples.

4. Contents

- 12 x 8 Microtiter strips
- 1 x Strip holder
- 1 x 12 ml Inactivated BLV antigen (Ready To Use)
- 1 x 12 ml Conjugated anti-BLV monoclonal antibodies (Ready to use)
- 1 x 12 ml Streptavidin conjugate
- 1 x 0,5 ml Positive control (freeze dried) (purple cap)
- 1 x 1,0 ml Negative control (freeze dried) (silver cap)
- 1 x 60 ml Wash solution (200x concentrated) (black cap), dilute in de-ionized water before use!
- 1 x 8 ml Substrate A (white cap)
- 1 x 8 ml Substrate B (blue cap)
- 1 x 8 ml Stop solution (yellow cap)
- 1 x Plastic cover seal
- 1 x User's manual

Supplies needed (not included)

- Round-bottomed microtiter plate
- Validated precision pipettes
- Pipette tips and clean containers/tubes (EVL)
- ELISA plate reader

5. Handling and storage of specimens

The kit should be stored at 4°C.

An open packet should be used within 10 days.

Samples may be used fresh or may be kept frozen below -20°C before use.

After first use ready-to-use controls and/or reconstituted controls should be aliquoted immediately and stored at -20°C.

Avoid repeated freezing and thawing as this increases non-specific reactivity.

6. Wash protocol

In ELISA's, un-complexed components must be removed efficiently between each incubation step. This is accomplished by appropriate washing. It should be stressed that each washing step must be carried out with care to guarantee reproducible inter- and intra-assay results. It is essential to follow the washing procedures outlined below. Washing may be done manually or with automatic equipment. Automatic washing equipment usually gives better result.

Manual washing

1. Empty each well by turning the microtiter plate upside down, followed by a firm vertical downward movement to remove the buffer.
2. Fill all the wells with 250 µl wash solution.
3. This washing cycle (step 1 and 2) should be carried out at least 5 times.
4. Turn the plate upside down and empty the wells with a firm vertical movement.
5. Place the inverted plate on absorbent paper towels and tap the plate firmly to remove any residual wash solution in the wells.
6. Take care that none of the wells dry out before the next reagent is added.

Washing with automatic equipment

When automatic plate washing equipment is used, check that all wells are aspirated completely and that the wash solution is correctly added, reaching the rim of each well during each rinsing cycle. The washer should be programmed to execute at least 5 washing cycles.

7. Preparations

- Before using the reagents needed, take them out of the kit and place them on the table for ± 15 min. at room temperature ($\pm 21^{\circ}\text{C}$) without exposing them to direct sunlight or (other) heat sources.
- Buffer, controls, standards and conjugates need to be shaken gently before use, in order to dissolve/ mix any components that may have precipitated. Gently tap the vials onto the table, so any fluid still retained in the cap falls back into the solution.
- If fluids need to be mixed into the test well, gently shake by tapping the wells with the fingers or re-suspend with the last pipette tip used for that particular well. Avoid contamination through spattering and prevent any fluid to enter inside the pipette itself.
- Place the reagents back at $4-8^{\circ}\text{C}$ immediately after use.

8. Test protocol Qualitative

Before starting this test read “preparations”

1. Open the packet of strips and take out the strips to be used. Cover the remaining strips with a part of the provided seal and store them at +4°C. and use them within 10 days.

Wash microtiter strip(s) with washing solution, according to washing protocol.

The washing solutions provided must be diluted 200x in aquabidest (5 mega Ohm) water !

Use validated precision pipettes and use a clean pipette tip **before** pipetting the buffer, control, samples, conjugate and substrate.

2. Reconstitute directly before use the **positive control** (purple cap) **in 0,5 ml aquabidest** (5 mega Ohm water), divide into aliquots, and store immediately at -20 °C until use, avoid freeze and thaw cycles.
3. Reconstitute directly before use the **negative control** (silver cap) **in 1,0ml aquabidest** (5 mega Ohm water), divide into aliquots, and store immediately at -20 °C until use, avoid freeze and thaw cycles.
4. Make a three-step dilution of the **positive control** (purple cap) **in ELISA buffer** (green cap) **starting undiluted → 1:3 → 1:9 → 1:27** in a **round-bottomed plate (not supplied)**.
Example:
 - Add 90µl positive control to the well **1A**.
 - Add 60µl ELISA buffer to all other wells **1B, 1C, 1D**
 - Transfer 30µl from well **1A** to well **1B**.
 - Mix well and transfer 30µl from well **1B** to the well **1C**.
 - Mix well and transfer 30µl from **1C** to the well **1D**.
 - Mix well and discard 30µl.
5. Add 60µl of the **negative control** to well **1E** of a round-bottomed plate (not supplied).
6. Add 60µl of **the sample** in **1F** a round-bottomed plate (not supplied).
7. Add **60 µl of inactivated BLV antigen** to all wells of the round bottom plate
8. Take 2 wells as **substrate controls** add only **120µl ELISA buffer** (green cap) to these wells.
9. Transfer 100µl of all dilutions to the virus-coated microtiter strips, including the substrate controls.
10. Seal and incubate for 30 min at 37°C.
11. Add **100µl conjugated anti-BLV monoclonal antibodies** to all wells (without emptying or washing the plate).
12. Seal and incubate for 60 min at 37°C.
13. Wash the strips according to the wash protocol ^{see sub 6}.
14. Add **100µl streptavidin conjugate** to all wells.
15. Seal and incubate for 30 minutes at 37°C.
16. Wash the strips according to the wash protocol ^{see sub 6}.

17. Mix equal parts of substrate A (white cap) and substrate B (blue cap) with gentle shaking. **Prepare immediately before use! Only prepare amount needed. Substrate can only be used for 1-2 hours after being mixed.**
18. Add **100µl substrate solution** to each well.
19. Incubate 10-20 min. in the dark (e.g. cover the wells with a sheet of paper). at room temperature (21°C.). Make sure the negative control does not become too dark.
20. Add **50µl stop solution** to each well; mix well.
21. Read the absorbency values immediately (within 10 min!) at 450nm by using an ELISA reader. **Use the substrate controls as blank.**

NB: if you pipet directly into the coated ELISA plate with only a small number of samples (<6), make sure the first dilution is done in round-bottom microtiter plate second step can be done directly in the coated Elisa plate.

9. Test protocol Quantitative

Before starting this test read “preparations”

1. Open the packet of strips and take out the strips to be used. Cover the remaining strips with a part of the provided seal and store them at +4°C. and use them within 10 days.

Wash microtiter strip(s) with washing solution, according to washing protocol.

The washing solutions provided must be diluted 200x in aquabidest (5 mega Ohm) water!

Use validated precision pipettes and use a clean pipette tip **before** pipetting the buffer, control, samples, conjugate and substrate.

2. Reconstitute directly before use the **positive control** (purple cap) **in 0,5 ml aquabidest** (5 mega Ohm water), divide into aliquots, and store immediately at -20 °C until use, avoid freeze and thaw cycles.
3. Reconstitute directly before use the **negative control** (silver cap) **in 1,0ml aquabidest** (5 mega Ohm water), divide into aliquots, and store immediately at -20 °C until use, avoid freeze and thaw cycles.
4. Make a three-step dilution of the **positive control** (purple cap) **in ELISA buffer** (green cap) **starting undiluted → 1:3 → 1:9 → 1:27** in a round-bottomed plate (not supplied).
Example:
 - Add 90µl positive control to the well **1A**.
 - Add 60µl ELISA buffer to all other wells **1B, 1C, 1D**.
 - Transfer 30µl from well **1A** to well **1B**.
 - Mix well and transfer 30µl from well **1B** to the well **1C**.
 - Mix well and transfer 30µl from **1C** to the well **1D**.
 - Mix well and discard 30µl.
5. Make a three-step dilution of the **negative control** (silver cap) **in ELISA buffer** (green cap) starting **undiluted → 1:3 → 1:9 → 1:27** in a round-bottomed plate (not supplied).
Example:
 - Add 90µl negative control to the well **1E**.
 - Add 60µl ELISA buffer to all other wells **1F, 1G, 1H**.
 - Transfer 30µl from well **1E** to well **1F**.
 - Mix well and transfer 30µl from **1F** to well **1G**.
 - Mix well and transfer 30µl from well **1G** to the well **1H**.
 - Mix well and discard 30µl.
6. Make a three-step dilution of **each sample in ELISA buffer** (green cap) starting **undiluted → 1:3 → 1:9 → 1:27** in a round-bottomed plate (not supplied).
Example:
 - Add 90µl of the sample to the well **2A and/or 2E**.
 - Add 60µl ELISA buffer to all other wells **2B, 2C, 2D and/or 2F, 2G, 2H**.
 - Transfer 30µl from well **2A and/ or 2E** to well **2B and/or 2F**.
 - Mix well and transfer 30µl from well **2B and/or 2F** to the well **2C and/or 2G**.
 - Mix well and transfer 30µl from **2C and/or 2G** to the well **2D and/or 2H**.
 - Mix well and discard 30µl.
7. Add **60 µl of inactivated BLV antigen** to all wells.
8. Take 2 wells as **substrate controls** add only **120µl ELISA buffer** (green cap) to these wells.
9. Transfer 100µl of all dilutions to the virus-coated microtiter strips, including the substrate controls.
10. Seal and incubate for 30 min at 37°C.

11. Add **100µl conjugated anti-BLV monoclonal antibodies** to all wells (without emptying or washing the plate).
12. Seal and incubate for 60 min at 37°C.
13. Wash the strips according to the wash protocol ^{see sub 6}.
14. Add **100µl streptavidin conjugate** to all wells.
15. Seal and incubate for 30 minutes at 37°C.
16. Wash the strips according to the wash protocol ^{see sub 6}.
17. Mix equal parts of substrate A (white cap) and substrate B (blue cap) with gentle shaking. **Prepare immediately before use! Only prepare amount needed. Substrate can only be used for 1-2 hours after being mixed.**
18. Add **100µl substrate solution** to each well.
19. Incubate 10-20 min. in the dark (e.g. cover the wells with a sheet of paper). at room temperature (21°C.). Make sure the negative control does not become too dark.
20. Add **50µl stop solution** to each well; mix well.
21. Read the absorbency values immediately (within 10 min!) at 450nm by using an ELISA reader. **Use the substrate controls as blank.**

10. Precautions

- Handle all biological material as though capable of transmitting infectious diseases.
- Do not pipette by mouth.
- Do not eat, drink, smoke or prepare foods, or apply cosmetics within the designated working area.
- TMB substrate (buffer B) is toxic by inhalation, through contact with skin or when swallowed; observe care when handling substrate.
- Do not use components past the expiry date and do not mix components from different serial lots.
- Optimal, results will be obtained by strict adherence to this protocol. Careful pipetting and washing throughout this procedure are necessary to maintain precision and accuracy.
- Each well is ultimately used as an optical cuvette. Therefore, do not touch the under-surface of the microtiter plate and protect it from damage and dirt.

11. Validation of the test

Qualitative:

- The results are valid if the following criteria are met:
 - The mean value (MV) of the measured OD value for the Positive Control (PC) diluted 1:2 must be ≤ 0.500 .
 - The MV of the measured OD value for the Negative Control (NC) diluted 1:2 must be ≥ 1.000 .

In case of invalid assays the test should be repeated after a thorough review of the instructions for use.

Calculation

Calculate the mean values (MV) of the measured OD for the Negative Control (NC) and the Positive Control (PC).

The ratio (S/N) of sample OD to mean OD of the negative control is calculated according to the following equation:

$$S/N = \frac{OD_{sample}}{MV \cdot OD_{NC}}$$

Quantitative:

In order to confirm appropriate test conditions the OD of the positive control, undiluted, should be ≤ 0.500 OD units (450nm) and give an endpoint titer of ≥ 9 .

The negative control, undiluted, should be ≥ 1.000 OD units (450nm) and give an endpoint titer of ≤ 3 .

12. Interpretation of the test results

This test can be used in 2 ways.




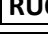


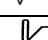


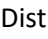
Qualitative: Positive – Negative

- A sample with the S/N ratio > 0.68 is negative.
 - Specific antibodies to bovine Leukaemia could not be detected.
- A sample with the S/N ratio ≤ 0.68 is positive.
 - Specific antibodies to bovine Leukaemia were detected.

Quantitative : End point titre

- The ELISA titre can be calculated by constructing a curve and using cut-off line(dilution undiluted - 1:3- 1:9 – 1:27 -1:88 – 1:176 etc. total 8 dilutions of 3 steps) OD on Y-as and titre on X-as.
Elisa titres can be calculated using as cut-off 2,5 times OD value of negative control at 1:3.

13. Symbols used with EVL ASSAYS

Symbol	English	Deutsch	Français	Español	Italiano
	Consult instructions for use	Gebrauchsanweisung beachten	Consulter les instructions d'utilisation	Consulte las instrucciones de uso	Consultare le istruzioni per l'uso
	European Conformity	CE-Konformitätskennzeichnung	Conformité aux normes européennes	Conformidad europea	Conformità europea
	In vitro diagnostic device	In-vitro-Diagnostikum	Usage Diagnostic in vitro	Para uso Diagnóstico in vitro	Per uso Diagnostica in vitro
	For research use only	Nur für Forschungszwecke	Seulement dans le cadre de recherches	Sólo para uso en investigación	Solo a scopo di ricerca
	Catalogue number	Katalog-Nr.	Numéro de catalogue	Número de catálogo	Numero di Catalogo
	Lot. No. / Batch code	Chargen-Nr.	Numéro de lot	Número de lote	Numero di lotto
	Contains sufficient for <n> tests/	Ausreichend für "n" Ansätze	Contenu suffisant pour "n" tests	Contenido suficiente para <n> ensayos	Contenuto sufficiente per "n" saggi
	Storage Temperature	Lagerungstemperatur	Température de conservation	Temperatura de conservación	Temperatura di conservazione
	Expiration Date	Mindesthaltbarkeitsdatum	Date limite d'utilisation	Fecha de caducidad	Data di scadenza
	Legal Manufacturer	Hersteller	Fabricant	Fabricante	Fabbricante
Distributed by	Distributor	Vertreiber	Distributeur	Distribuidor	Distributore
Content	Content	Inhalt	Conditionnement	Contenido	Contenuto
Volume/No.	Volume / No.	Volumen/Anzahl	Volume/Quantité	Volumen/Número	Volume/Quantità

The entire risk as to the performance of these products is assumed by the purchaser. EVL shall not be liable for indirect, special or consequential damages of any kind resulting from use of the products In case of problems or questions contact EVL.